Attenuation table* - Loop earplugs

Frequency (Hz)	63	125	250	500	1000	2000	4000	8000
Mean attenuation (dB)	15.5	17.0	15.7	14.3	19.0	23.2	30.4	20.4
Standard deviation (dB)	5.5	3.8	5.0	2.8	3.5	1.8	4.5	5.7
Assumed protection value (dB)	10.0	13.2	10.7	11.5	15.5	21.4	25.9	14.7

CE 1974 | EN 352 Tested according to EN 2:2002-352 and yearly control by PZT GmbH, Bismarckstr. 264, D26389- Wilhelmshaven (NB 1974)

*Tested with silicone ear tips.

Foam ear tips are not certified as hearing protection according to (EU)425/2016

SNR = 18 dB H = 19 dB M = 15 dB L = 13 dB

EAR TIP SIZE: S = 11 mm M = 12mm L = 13mm

Loop bvba declares that the PPE Loop earplugs is in compliance with the Regulation (EU) 2016/425 and certified by PZT GmbH. The full text of the EU Declaration of Conformity is under following link: loopearplugs.com/doc

Loop bvba, Vorstermanstraat 2000, 11 Antwerpen, Belgium
hello@loopearplugs.com

EN Loop protects your ears against hearing loss, damage and tinnitus. Wear them at concerts, events, against noise, etc. Insert by taking the earplug between your thumb and index finger. To ease insertion, pull the top of your ear. Gently push and twist until the ear tip seals your ear canal completely. Remove Loop with a slow twisting motion to gradually break the seal.

Wipe clean with a damp cloth. Do not use chemicals, sharp objects or immerse in water. More information can be requested from the manufacturer. Always wear in noisy surroundings. Always store in carry case. Inspect regularly for serviceability. Keep out of reach of children. Wear and maintain according to instructions. Protection will be impaired if recommendations are not followed. Loop is not liable for damage by misuse or loss of hearing.

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